



THE UNIVERSITY OF UTAH

**OCCUPATIONAL AND ENVIRONMENTAL
HEALTH AND SAFETY**

Monthly Maintenance Check Procedure for the Zoll AED Plus

- Check for the green check (✓) in the status indicator window showing that the AED Plus is ready to use.
- Verify that electrodes pads are within their expiration date.
- Verify that batteries are within their expiration date. Typically 5 years from date of installation.

Note: there is not an expiration date printed on the batteries. Only the installation date as noted on the exterior of the case.

- Verify that electrodes and cables are undamaged and properly connected to the AED.
- Verify that supplies are available for use (razor, mask, gloves, extra batteries)
- Verify that the unit is clean, undamaged and free from excessive wear. Contact OEHS at 801-581-6590 to report damage.
- Turn the AED on and off to verify function. The green check (✓) should remain lit in the status indicator window.
- Record the monthly inspection on the inspection tag attached to the AED unit.

Report use of the AED to OEHS at 801-581-6590 as soon after the use as possible, OEHS will re-supply the AED.